


Replace wax guard 'Sentry II'

Hearing instruments of type CIC or MC instrument are usually equipped with a wax guard called "Sentry II". Please verify this with your hearing care practitioner.


Sentry II wax guards are available in a set, containing red guards, blue guards, and a dedicated tool for changing them.

Use red wax guards for right instruments and blue guards for left instruments.


- To remove the wax guard from your hearing instrument, slide the forked side of the tool under the wax guard and pull it upwards.

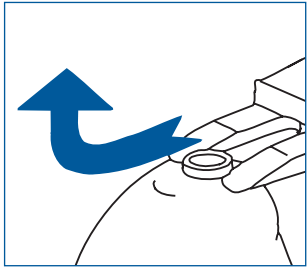

- Pick up a new wax guard from the front side of the card by using the other 'nub' end of the tool. The large red and blue arrows on the card indicate the front side. Slide the wax guard to the side, through the card.

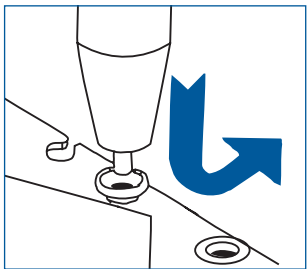

- Insert the wax guard into the sound outlet of the hearing instrument.

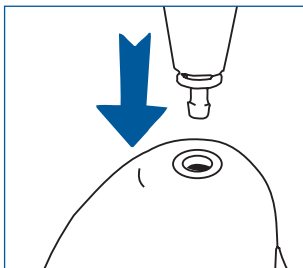

- Remove the tool and press the wax guard down with your thumb to secure it.

